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Dischi Girevoli Turn Up.

Codice	Ø (cm)	Portata (Kg)
03.04.5082	38.5	180

**Destinazione d'uso**

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Use

The Transfer Systems are intended for facilitating the manual handling operations of patients through thrust, traction, sliding maneuvers. They have been designed for use in hospitals, nursing homes and healthcare facilities and for home care.

Description

The TURN UP is made of resin with a non-slip base. The contact surface is covered with an anti-slip material. It is designed for patients who are able to stand up, but cannot move their legs easily. The operator, thanks to the disk, can transfer the patient with the minimum effort.

The disc has to be placed on the floor and the patient's feet must be both on it. Then, the operator will have to help the patient to stand up from the bed and move the patient by rotating the disk with a foot towards the desired direction.

Materials	
Plate	Resin
Coating	Antislip Fabric
Maintenance and Cleaning	
Disinfection is possible with 5% diluted disinfectants even with alcohol base	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr.Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.a.S.
Directive 93/42	Class I
ISO Code	12.31.03.006
CND Code	Y123099
RDM Code	1167517/R