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Deambulatore 4 Ruote.

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Product Name

4 Wheels One Botton Folding Walker.

Code	Length	Width	Height	Weight Capacity	Front Wheel	Rear Wheel	Net Weight	Carton Size
Measures in Cm								
01.06.300	57.5	66	82.5/92.5	120	10 (4")	10 (4")	3.4	56x16x69

Use

The walkers and rollators are indicated in the walk rehabilitation for elderly or subjects with functional limitations of various kinds. They offer stability, ease of use and improve the sense of balance.

This aid is designed for a single user, in indoor or outdoor environments according on models.

Description

The 4 WHEELS ONE BUTTON FOLDING WALKER has a foldable aluminum structure that is height adjustable. It has two rotating front wheels within the outline and two rear twin wheels. The front wheels have a directional **locking system**, by operating a lever you can switch from fixed wheels (lever upwards) to swivel wheels (lever downwards).

Furthermore, it is equipped with one horizontal stabilizing bar and four legs with anti-slip tips. The folding can easily be done using the locking pin located at the center of the upper bar, the knobs are ergonomic and padded for a more comfortable grip. Rear glide walker brakes

Materials	
Frame	Aluminum
Knobs	Polypropylene
Wheels	PVC
Maintenance and Cleaning	
Use a damp cloth with mild soap, the water temperature not above 30°C	
Dry well before use	
If necessary, disinfect the product using a common disinfectant detergent	
Do not use water and / or steam jet washing machines	
Must not use acid, alkaline or solvent products such as acetone	
Storage and Disposal	
Single-user Product	
Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr.Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.a.s.
Directive 93/42	Class I
ISO Code	12.06.06.006
CND Code	Y120606
RDM Code	1706666/R