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Use

The gel positioners are devices that are used to reach an optimal patient position on the operating table. Many differentes models are available in various shapes and sizes; all fully compatible with human tissues. Some of the main features are: biocompatibility, comfort and extraordinary mechanical resistance.

Description

The PRONE HEADREST is made of an amorphous polyurethane gel with a density of 0.99 gr/cm³. Thanks to its particular ergonomic design, it can be used as a neck and head support during all types of surgery, with an easy access for intubation.

MaterialsAmorphous Polyurethane Gel with a Density of 0.99 gr/cm³**Maintenance and Cleaning**

The gel positioners can be easily cleaned with any cleaning agent and / or disinfectant for the operating room

Avoid immersing the gel positioners in the detergent and / or disinfectant for a long time

Storage and Disposal

Keep the positioners, if possible, on a level surface

Avoid placing heavy or sharp objects on the positioners

Keep the gel temperature between 50 ° and -18 °

Keep away from sources of light, moisture and heat

Dispose of the device in compliance with current regulations on environmental protection and separate waste collection

Package and Nr.Items

Box 1 Piece

Warranty

European 24 months for manufacturing defects with the exception of consumable parts

Product References

Manufacturer Demarta Virginio S.r.l.

Directive MDR 2017/745 Class I

CND Code Y033399

RDM Code 90736/R