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Product Name

Adjustable chest positioner with waterproof and fireproof polyurethane lining.

Code	Dimensions (cm)	Hole measurements (cm)
05.00.AP6001	46 X 90 X 12	26/46 x 30

Use

The Adjustable Chest Positioner is used to ensure maximum comfort for the patient undergoing spinal surgery.

Description

The Adjustable Chest Positioner has a closed-cell foam support base with a viscoelastic foam support surface. The outer shell has a waterproof and breathable polyurethane lining with a non-slip base and 3-sided zipper. The structure is equipped with two inserts which, when removed, bring the length from 70 to 90 cm. The rectangular hole inside the positioner is 26/46 x 30 cm in size.

The positioner is equipped with an additional reducer inside with dimensions 43 x 27.5 cm, and the hole inside is 30 x 14.5 cm.

The positioner is equipped with a space to rest your face during spinal surgery.

Materials	
External Covering Docking station Support surface	Polyurethane lining with non-slip base
Docking station	Closed cell foam
Support surface	Viscoelastic foam
Maintenance and Cleaning Positioner	
The gel positioners can be easily cleaned with any cleaning agent and / or disinfectant for the operating room	
Avoid immersing the gel positioners in the detergent and / or disinfectant for a long time	
Lining Maintenance and Cleaning	
	Wash Max 95° C
	Do not iron
	Do not bleach
	Do not use the dryer
Storage and Disposal	
Avoid placing heavy or sharp objects on the positioners	
Keep away from sources of light, moisture and heat	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr.Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	Demarta Virginio S.r.l.
Directive MDR 2017/745	Class I
ISO Code	04.33.09
CND Code	Y033399
RDM Code	90736/R