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Product Name

Cylindrical Protection Polymoove.

Code	Dimensions
03.05.POL40	80 x 8 cm
03.05.POL42	100 x 18 cm
03.05.POL45	30 x 8 cm

Use

Polymoove Positioners are a valid aid for healthcare professionals for patient positioning. They are designed to give comfort, support, stability and protection of vulnerable areas. The positioners adapt to the morphology of the body excellently thanks to the composition of granulated foam spheres. The various shapes are easily combined with each other and ideal for an efficient anti-decubitus prophylaxis. The cylindrical positioner is used as a support for the neck and as a support for the lower and upper limbs.

Description

The POLYMOOVE CYLINDRICAL POSITIONER is internally composed of granulated foam contained in a protective cotton casing. The Polymoove filling material offers optimal patient support by helping to distribute body weight in a lying or sitting position. The cushion is covered with an external cover in Obatex fabric, removable thanks to the zip on the side. The fire retardant, breathable, bi-elastic, waterproof and antibacterial OBATEX lining is made of polyurethane and polyester, sterilizable in an autoclave. The Polymoove does not deform after being washed and dried in the washing machine, the positioning cushions can be used for many cycles without altering their shape. Their low moisture absorption rate facilitates cleaning and environmental sustainability.

Materials	
Padding	Espanso Granulato in Polipropilene (EPP)
Protective Overwrap	Cotton
External cover	Obatex
External Cover Maintenance and Cleaning	
	Water temperature not above 95°C or 200°F
	Do not bleach
	Do not iron
	Dryclean with any solvent
	Tumble Dry Max temperature 60°C or 140°F
We recommend washing the cover at least once a year	
Positioner Maintenance and Cleaning	
	Water Temperature not Above 60°C or 140°F
	Do not Bleach
	Maximum Temperature 150°C or 300°F
	Dryclean with any Solvent
	Tumble Dry Medium Heat
We recommend washing the positioner at least once a year	
Storage and Disposal	
Single-user Product	

Keep away from heat, sparks, open flame or other sources of ignition	
Dispose of the device in compliance with current regulations on environmental protection and separate waste collection	
Package and Nr.Items	
Box	1 Piece
Warranty	
European	24 months for manufacturing defects with the exception of consumable parts
Product References	
Manufacturer	OBA AG
Authorized Distributor	Demarta Virginio S.r.l.
Directive MDR 2017/745	Class I
CND Code	Y060399
RDM Code	225942/R